

RACE EVO

INFORMATION CARD

Standard sequence for most cases finishing with 30/.04 or 25/.06 depending on your preferred therapeutic approach.

800-1000 (1.5 Ncm)




RE1 - 15/.04



RE2 - 25/.04




RE3 4% - 30/.04 OR RE3 6% - 25/.06

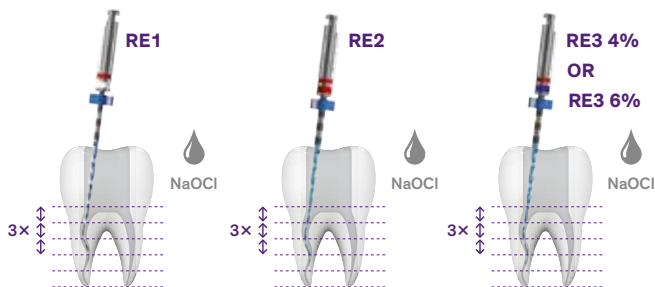

- Create straight-line coronal and radicular access.
- Use an ISO 010 hand file to explore the canal.
- Determine the working length (WL).

- Use **RACE EVO** files with gentle 2-3 mm strokes, applying very light apical pressure and allowing the file to progress passively along the canal. After 3 strokes, remove and clean the file, then irrigate ; recapitulate.
 - Use **RE1** to perform glide path until the WL is reached.
 - Start the shaping procedure by using **RE2** until the WL is reached.
 - Use **RE3** for the shaping and final apical preparation.
- Select the final shaping file depending on your preferred therapeutic approach (4% or 6%).*

- Once the canal has been shaped, proceed with the cleaning of the canal.
- Obturate by selecting the appropriate GP point & sealer.

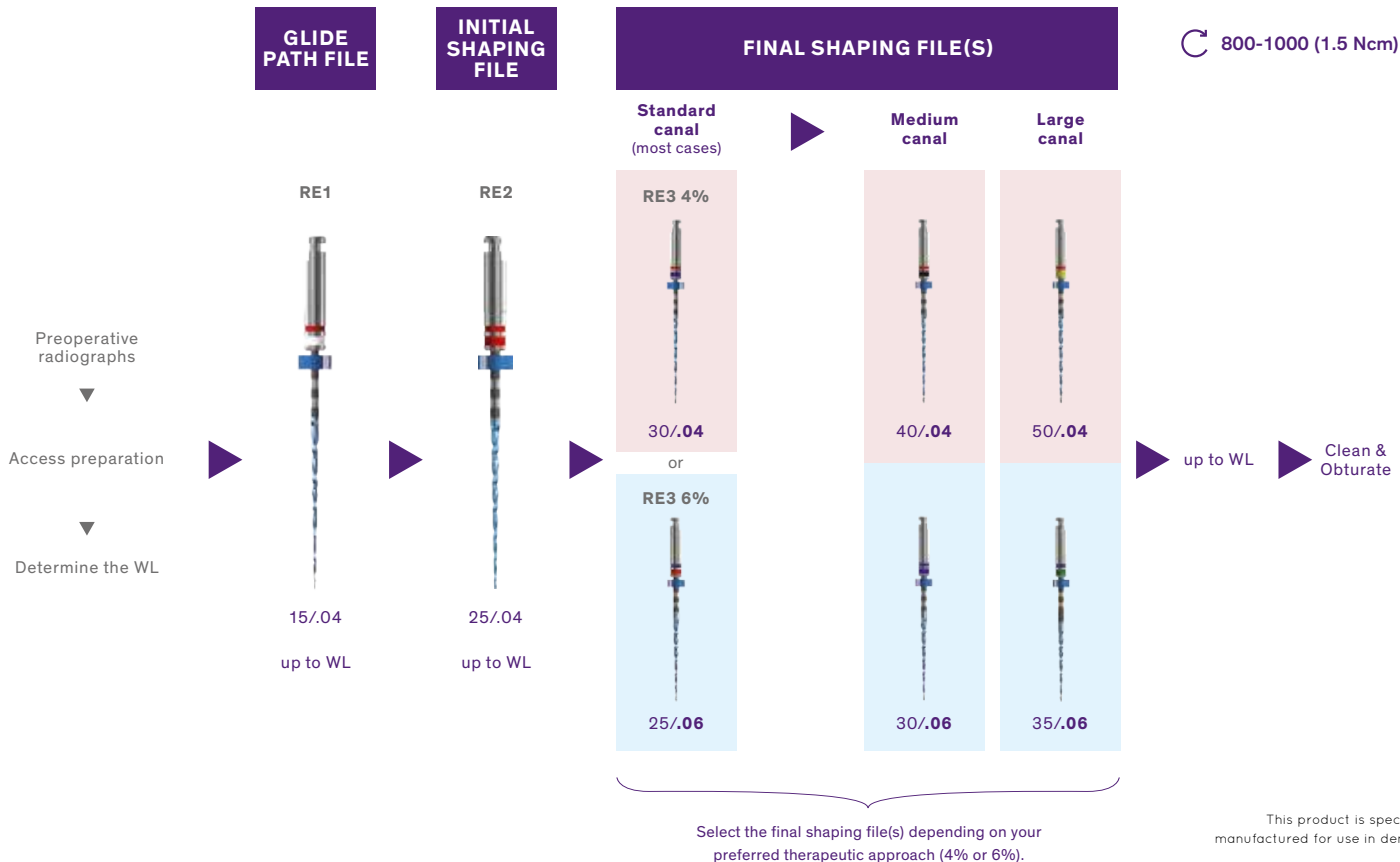

W597/5/281020