Davis Schottlander & Davis Ltd.  
Letchworth, Herts, UK  
Tel: +44 (0)1462 480848  
[www.schottlander.com](http://www.schottlander.com)

**schottlander**

en

# Matchmaker CTE Buffer

## Instructions for Use

### INDICATIONS

Matchmaker CTE Buffer is used as thin layer between the alloy and first opaque. It blocks the metal oxide under the opaque and neutralises the differences in coefficients of thermal expansion (CTE). Matchmaker CTE Buffer does not increase the bond but allows a wider range of CTE compatibility

### CONTRA INDICATIONS

Do not use on alloys not designed for porcelain fused to metal.

### PREPARING THE METAL FRAMEWORK

Prepare the metal framework and carry out oxidation in accordance with the alloy manufacturer's instructions.

### APPLICATION

Stir every time before use. Using a clean Propaque opaque brush, apply in as thin a layer as possible so it masks the framework completely.

**NOTE.** Do not re-moisten once the applied Matchmaker CTE Buffer has dried.

To avoid the paste drying out, always replace the lid immediately after use. If re-moistening is required, then use enliven regeneration liquid.

The correct appearance after firing is slightly yellow with a shiny appearance (this colour will vary depending on the composition of the alloy used).

The subsequent opaque firing should never be above that of the Matchmaker CTE Buffer.

### FIRING

Start temperature	Drying time	Temperature rise	Vacuum	High temperature	Hold time
550°C	6 minutes	80°C/min	Yes	980°C	1 minute

### OPAQUE APPLICATION

Fire Propaque or enliven MC paste opaque in accordance with the instructions for use. Note that when using Matchmaker CTE Buffer on non-precious alloys that the first opaque firing no longer has to be increased by 20°C.

## STORAGE

Store in a cool, dry place.

## HEALTH & SAFETY

Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Use in a well-ventilated area. Wash off any material that comes into contact with skin. Wash using soap and water.

If the material comes into contact with eyes, flush with large volumes of water for at least 20 minutes. Seek medical advice.

**Ingestion:** Wash out mouth thoroughly. Seek medical advice.

**Inhalation:** Remove the person to fresh air. If symptoms persist seek medical advice.

## LOT NUMBERS

The lot number is shown on all packs.

This product is specifically formulated for use in dentistry.

Matchmaker is a internationally registered trade mark of Davis Schottlander & Davis Limited.

Where this product is shown as having been certified as a medical device in the European Union under the Medical Device Directive 93/42/EEC by SGS CE1639, this is exclusively for the indication(s) shown in the above Instructions for Use. Other non-medical uses ascribed to this device are not within the scope of CE certification, and users should be aware that product performance and/or safety has not been evaluated by SGS for those purposes.

**Schottlander (IRL) Ltd.**

Suite 11, Parklands Office Park, Southern Cross Road,  
Bray, Co Wicklow, Eire.

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Matchmaker è un marchio registrato a livello internazionale di proprietà di Davis Schottlander & Davis Limited.