



Quicktemp Cosmetic Glaze & Bond

Instructions for Use

INDICATIONS

Quicktemp Cosmetic Glaze & Bond is suitable for use as a glazing agent to prevent staining of and add a gloss finish to bis-acrylic temporary crowns and bridges. It is also suitable for use as a bonding agent for the repair of bis-acrylic temporary restorations.

CONTRA-INDICATIONS

Do not use with patients with known allergies to acrylates.

APPLICATION & CURING

Use as a bonding or repair material.

Apply a thin, air free layer of Quicktemp Cosmetic Glaze & Bond to the area to be bonded. Allow to stand for 20 seconds and cure with a LED or Halogen light unit for 30 seconds.

Note: If a chemical cure repair material is to be used, cure the Quicktemp Cosmetic Glaze & Bond for 60 seconds.

Apply fresh Quicktemp or Light Cured Composite material onto the treated surface, cure according to manufacturers instructions for the product used.

For use as a Glaze.

Coat the temporary with Quicktemp Cosmetic Glaze & Bond and light cure for 90 seconds depending on the curing light used. Light curing must take place within 40 seconds of application to ensure enough solvent remains to prevent oxygen contact to the site.

Replace the lid onto the bottle immediately after use to prevent evaporation. Failure to replace the lid promptly after each use may affect physical properties.

Note: Quicktemp Cosmetic Glaze & Bond contains multifunctional acrylates, methyl methacrylate, catalyst, stabilisers and additives.

STORAGE

Quicktemp Cosmetic Glaze & Bond is light sensitive. Use and store away from bright light and replace cap as soon as possible.




HEALTH AND SAFETY

Avoid contact of the liquid with skin especially with people with known allergies in particular to methacrylates.



Warnings: H225 Highly flammable liquid and vapour. H315 Causes skin irritation. H317 May cause an allergic skin reaction. H319 Causes serious eye irritation. H335 May cause respiratory irritation. H412 Harmful to aquatic life with long lasting effects.

Precautions: P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P261 Avoid breathing dust/fume/gas/mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P302+352 IF ON SKIN: Wash with plenty of water. P501 Dispose of waste according to applicable legislation.

For further information download a Safety Data Sheet from  www.schottlander.com

LOT NUMBERS

The lot number and expiry date are shown on all outer containers.

This product is specifically formulated for use in dentistry.

Quicktemp is an internationally registered trade mark of Davis Schottlander & Davis Ltd.

Where this product is shown as having been certified as a medical device in the European Union under the Medical Device Directive 93/42/EEC by SGS CE1639, this is exclusively for the indication(s) shown in the above Instructions for Use. Other non-medical uses ascribed to this device are not within the scope of CE certification, and users should be aware that product performance and/or safety has not been evaluated by SGS for those purposes.

