



Davis Schottlander & Davis Ltd.
Letchworth, Herts, UK
Tel: +44 (0)1462 480848
www.schottlander.com

schottlander

Doric Toughened Wax

Instructions for Use

Baseplate Wax Soft. Type 2 Class 1.
Complies with EN ISO 15854:2021.

Model Forming

Take a sheet of wax and cut length required, soften the sheet of wax over a bunsen flame and adapt it onto the plaster model. Ensure the wax is fitting the model, allow to cool and then trim/carve the excess wax away using a heated knife.

Other uses

Take a sheet of wax and soften over a bunsen flame and roll the wax into a block for bite rims.

Also can be suitable for bite registration using a strip of wax of suitable width.

Safety

For full information on safety please visit our website www.schottlander.com Download Centre, Safety Data Sheets.

This product is specifically formulated for use in dentistry.

Doric is an internationally registered trademark of Davis Schottlander & Davis Ltd.



W5970/0621