# matchMaker MC

## Instructions for Use

### Indications

Matchmaker MC is a veneering ceramic porcelain for bonding to metal crowns and bridges in accordance with EN ISO 9693-2019.

### Contra Indications

Do not use without a metal sub-frame. Not suitable for use with low fusing porcelains, PJC materials or with some non-precious alloys. Do not use with a sub-frame other than described in Alloys below.

### Alloys

Non Precious, Semi Precious & Precious Alloys with coefficient of expansion between  $13.8 - 14.9 \times 10^{-6} \text{°C}^{-1}$  may be used. Prepare your copings and where indicated, degas and/or oxidise to the alloy manufacturer's directions for use.

### Opaque Applications

Extrude a small amount of Propaque onto a clean palette and stir before application. Apply in one direction only and never thin with water.

When using Matchmaker MC powder opaque, mix with Matchmaker opaque liquid to the desired consistency.

With a Propaque brush apply two even thin layers completely masking the coping.

When fired, the 1st Opaque is shiny with a partial opaque coverage to the surface. After firing the second opaque has a textured surface and a slight sheen.

### Dentine & Incisal Build Up

Apply a thin layer of Matchmaker Enhancer over the entire fired opaque and finish with an irregular mamelon shape to the incisal area. Build up the Matchmaker Dentine to full contour and cut back allowing space for the enamel by reducing the buccal surface two thirds down from the incisal edge towards the cervical. At the same time reduce the incisal dentine length by approximately 1mm. Shape the incisal dentine to replicate the mamelon structure, leaving the incisal edge irregular and thin at the edge.


Apply a thin layer of Matchmaker Clear over the buccal dentine extending slightly towards the incisal. Overlay the enamel clear with the chosen Matchmaker Enamel shade (see Figure 1). To allow for shrinkage, slightly extend both the enamel at the incisal edge and the dentine and enamel at the contacts.

### Small Corrections

Ensure the surface is clean before making a correction. If the previously mixed material on the mixing slab has dried out, only re-hydrate with deionized water.

### Glaze & Stain

After final shaping ensure the surface is clean and dry. Apply a thin wash of glaze & stain liquid to the surface and compare the shade to your shade guide. Slight shade correction can be made by using enliven Universal Stains.

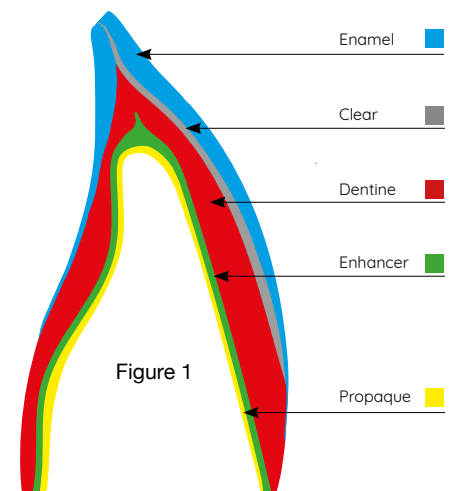## Overview of Matchmaker MC & Propaque Firing Instructions

Furnace temperatures vary. Adjust high temperature to give appearance in firing chart.

Matchmaker MC	Start Temp °C	Minimum Drying Time	Temp Rise °C / Min	Vacuum	High Temp °C	Hold Time Without Vacuum	Appearance
Degassing & Oxidation							
Refer to the alloy manufacturer's instruction							
Propaque 1st firing	550	8 min	80	Yes	980	1 min	Slight sheen
Propaque 2nd firing	550	8 min	80	Yes	960	1 min	Textured, slight sheen
Powder Opaque 1st firing	580	2 min	80	Yes	980	1 min	Textured, slight sheen
Powder Opaque 2nd firing	580	4 min	80	Yes	950	1 min	Textured, slight sheen
Shoulder 1st firing	600	2 min	80	Yes	950	1 min	Slight sheen
Shoulder 2nd firing	600	4 min	80	Yes	940	1 min	Slight sheen
Dentine, Enhancer, Enamel etc 1st firing	580	6 min	60	Yes	930	1 min	Textured, slight sheen
Dentine, Enhancer, Enamel etc 2nd firing	580	6 min	60	Yes	920	1 min	Slight sheen
Glaze with Glaze Powder	580	6 min	60	No	920	1-2 min	Glaze
Glaze without Glaze Powder	580	6 min	60	No	930-940	1-2 min	Glaze depending on requirements

Precious alloys generally require the opaque temperatures to be fired 20°C lower.

### STORAGE

Store Matchmaker MC powders between 4°C and 29°C and Propaque between 15°C and 23°C. Immediately after each usage replace all caps.

### HEALTH AND SAFETY

Keep away from eyes since the granular nature of the powder or paste may cause irritation.

### LOT NUMBERS

The lot number is shown on all packs.

This product is specifically formulated for use in dentistry.

Matchmaker and Propaque are internationally registered trade marks of Davis Schottlander & Davis Limited.

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