

Schottlander Dental Dam Frame

Instructions for Use

The Schottlander Dental Dam Frame is specifically designed to afford maximum cheek and lip retraction. The overall width and curvature of the frame has been increased to give exceptional stability for all dental procedures. The vertical arms are half an inch shorter to avoid the eye area. There are eleven tines to allow for smooth and easy attachment of the dental dam.

NOTE: Due to the unique features of this frame the dental dam must be applied over the frame. To do this, hold the frame under the dam, stretch the rubber over the eleven tines and smooth it tautly over the frame for maximum retraction and stability.

Schottlander Dental Dam Frames are autoclavable, radiolucent and unbreakable under normal usage.

For use in dentistry.

Davis Schottlander & Davis Ltd.
Fifth Avenue, Letchworth Garden City, Herts SG6 2WD England
Tel: +44(0)1462 480848 www.schottlander.com



W3595/0409