

SCHOTTLANDER MOULD CUTTING DIAMOND DISCS (Code 0138)

DIRECTIONS FOR USE

These discs are for use in sectioning plaster or investment models and rings. The optimum speed is 10–15,000 rpm. When brought into contact with a model or ring the disc must have a minimum speed of 10,000 rpm and a maximum of 20,000 rpm.

Instructions for use

1. Insert the shank into the handpiece as deeply as possible.
2. Work with the minimum pressure possible and in accordance with the above speed instructions.
3. To avoid flexing of the disc, steady your working hand on a suitable support such as the bench top or benchpeg.
4. Always wear eye protection against flying objects and employ an extraction system to remove dust and small particles.
5. Where necessary use a white cleaning stone (at right angles to the direction of running) to restore the cutting efficiency of the disc.

Not for use in the mouth.

If the bur has become bent, through the handpiece being dropped or for any other reason, discard the bur immediately.

W3280/0605

Davis Schottlander & Davis
Ltd, Letchworth, Herts,
England +44 (0)1462 480848
www.schottlander.com

schottlander