











Schottlander Amalgam Removal Bur Kit

Product
Code 913

This kit provides a series of burs which will aid rapid removal of amalgam restorations and remove the fragments which are found in undercuts and irregular areas of the cavity wall and floor. It contains both medium grit Schottlander Multi-Layered Gold Diamonds, Schottlander Multi-Layered Fast-Cut Cross Cut Tungsten Carbides and Schottlander Gold Fast-Cut Tungsten burs to suit the conditions encountered. Effective removal with reduced tooth tissue loss may be achieved by cutting down to the depth of the amalgam and then cutting along the tooth/restoration margin at the expense of the amalgam. The restoration may then be displaced using hand instruments.

Pattern Number	Quick Order Code	Region of Use/Uses
Pear 830L 012 4 830L 014 5	M063  M064 	All regions. Used for fast removal with rapid clearance along margin of restoration where amalgam remains in undercuts.
Flat End Cylinder with Rounded Corners 835R 012 4 835R 014 4	M422  M423 	All regions. Used for fast bulk removal with rapid clearance along tooth restoration interface.
Cross Cut Dome Ended Taper 2157 010 4 2158 012 4	2170  2171 	All regions. Used for precise removal of amalgam fragments.
Gold Fast-Cut Tungsten GT1 Round 2G 010 4G 014 6G 018 GT2 Inverted Double Cone 35G 010 1.2	GT-01  GT-02  GT-03  GT-12 	— All regions. Universal burs. May be used for initial access to determine depth of restorative. For removal of amalgam fragments from undercuts.

Davis Schottlander & Davis Ltd.

Fifth Avenue, Letchworth Garden City, Hertfordshire, SG6 2WD Tel: +44 (0)1462 480848 www.schottlander.com



WG515/1108