

SCHOTTLANDER AUTOCLAVABLE BITE PLANE

For use in the setting of the occlusal plane of the teeth into acceptable anatomic and aesthetic positions.

This product is made from UDEL polysulfone and is supplied non-sterile.

Steam autoclave (maximum 134°C) in accordance with autoclave manufacturers instructions before each time of use.

DIRECTIONS FOR USE

Place the intraoral part of the bite plane onto the occlusal surface of the record rim. The extra oral extension can then be used by the clinician to judge the relationship of the plane to the facial guideline.

Adjust the rim so that the occlusal plane is parallel to the inter-pupillary line or is at right angles to the long axis of the patient's face. For the anterior posterior orientation adjust the plane so that it is parallel to the ala-tragal line.

When viewed from the front the Schottlander Autoclavable Bite Plane should be positioned so that it is level with the line of the pupils of the eyes.

This product is specifically designed for use in dentistry.

Davis Schottlander & Davis Ltd.
Fifth Avenue, Letchworth Garden City, Hertfordshire SG6 2WD England
Telephone +44(0)1462 480848 www.schottlander.com

