







Schottlander Composite Tungsten Finishing & Polishing Bur Kit





Product
Code 948

This kit contains a range of Schottlander Tungsten Carbide multi-bladed burs which may be used for initial reduction and contouring of composite restoratives after polymerisation as well as for subsequent polishing. The twelve bladed burs will permit bulk removal, modify the shape of the restoration and produce a relatively smooth surface. The thirty bladed burs produce a final smooth restoration. The profiles of the thirty bladed burs match those of the twelve bladed burs permitting the trimmed surface to be polished using the same bur configuration to optimise the prepared restorative surfaces while the bur profiles permit finishing to all accessible surfaces of the restoration.

TUNGSTEN FINISHING (12 Bladed)

Pattern Number	Quick Order Code		Region of Use/Uses
023 4	7408		All regions. Palatal and occlusal contouring.
009 3.5	7114		All regions. Fine fissures and interproximal surfaces.
018 -	7006		All regions. Universal bur.
014 8.5	7714		All regions. Axial and interproximal reduction (Long reach).
010 5.5	7214		All regions. Axial and narrow interproximal reduction (Intermediate reach).
012 4.5	7903		All regions. Fissure refinement and contouring occlusal surfaces (Short reach).

TUNGSTEN POLISHING (30 Bladed)

Pattern Number	Quick Order Code		Region of Use/Uses
018 4	9406		All regions. Smoothing palatal and occlusal surfaces.
014 8.5	9714		All regions. Axial and interproximal finishing (Long reach).
010 5.5	9214		All regions. Axial and interproximal finishing (Intermediate reach).
012 3.5	9103		All regions. Axial finishing (Short reach).

Tooth height and the extent to which patients can open their mouths creates the need for long and short reach burs to carry out finishing at different sites. It is important to bear in mind that the shorter burs reduce wear on handpiece bearings and this in turn will extend handpiece life.

Davis Schottlander & Davis Ltd. Fifth Avenue, Letchworth Garden City, Hertfordshire, SG6 2WD Tel: +44 (0)1462 480848 www.schottlander.com



W35050912

Products for better dentistry

schottlander