

Schottlander Finishing Diamonds Bur Kit



















Product Code 941

This bur kit provides a comprehensive set of Schottlander Multi-Layered Gold Finishing Diamonds in three levels of fineness for finishing tooth preparations and composite restoratives. They should be used in the following sequence:- fine, extra fine and ultra fine.

Fine grit (45µm) burs provides initial finishing and smoothing (red ring). ●

Extra fine grit (20µm) burs provides intermediate finishing, suitable for final finishing of microfine and nanoparticle composites (yellow ring). ●

Ultra fine grit (10µm) burs provides a final finish for hybrid and heavily filled composites (white ring). ○

Pattern Number	Quick Order Code	Region of Use/Uses
Round 801 018 1.8	F004  XF004  UF004 	All regions. Universal bur for finishing fissures and characterising occlusal surfaces.
Football 368 023 5 379 023 5	F052  XF052  UF052  F054  XF054  UF054 	Anterior and posterior. Cingulum and palatal finishing. Occlusal contouring of fissures.
Flame 860 012 5 863 014 10	F389  XF389  UF389  F395  XF395  UF395 	All regions. Fine finishing at margins. Posterior. Fine finishing at margins where long reach required.
Round Ended Cylinder 880 012 6	F410  XF410  	All regions. Finishing of large areas with low curvature such as labial surfaces or axial walls.

Davis Schottlander & Davis Ltd. Fifth Avenue, Letchworth Garden City, Hertfordshire, SG6 2WD Tel: +44 (0)1462 480848 www.schottlander.com



Products for better dentistry

schottlander

W3520/0907