



Davis Schottlander & Davis Ltd.  
Letchworth, Herts, UK  
Tel: +44 (0)1462 480848  
[www.schottlander.com](http://www.schottlander.com)



DFS-Diamon GmbH  
Ländenstraße 1  
D-93339 Riedenburg



REF 941D

**schottlander**



















# Schottlander Finishing Diamonds Bur Kit

This bur kit provides a comprehensive set of Schottlander Multi-Layered Gold Finishing Diamonds in three levels of fineness for finishing tooth preparations and composite restoratives. They should be used in the following sequence:- fine, extra fine and ultra fine.

Fine grit (45µm) burs provides initial finishing and smoothing (red ring). ●

Extra fine grit (20µm) burs provides intermediate finishing, suitable for final finishing of microfine and nanoparticle composites (yellow ring). ●

Ultra fine grit (10µm) burs provides a final finish for hybrid and heavily filled composites (white ring). ○

Pattern Number	Quick Order Code	Region of Use/Uses
<b>Round</b> 801 018 1.8	<b>F004</b>  <b>XF004</b>  <b>UF004</b> 	All regions. Universal bur for finishing fissures and characterising occlusal surfaces.
<b>Football</b> 368 023 5  379 023 5	<b>F052</b>  <b>XF052</b>  <b>UF052</b>  <b>F054</b>  <b>XF054</b>  <b>UF054</b> 	Anterior and posterior. Cingulum and palatal finishing. Occlusal contouring of fissures.
<b>Flame</b> 860 012 5  863 014 10	<b>F389</b>  <b>XF389</b>  <b>UF389</b>  <b>F395</b>  <b>XF395</b>  <b>UF395</b> 	All regions. Fine finishing at margins.  Posterior. Fine finishing at margins where long reach required.
<b>Round Ended Cylinder</b> 880 012 6	<b>F410</b>  <b>XF410</b>  <b>UF410</b> 	All regions. Finishing of large areas with low curvature such as labial surfaces or axial walls.