

## Doric Gingival Retraction Paste Instructions for Use

### Introduction

Doric Gingival Retraction Paste is a retraction agent for sulcus expansion. It also has an astringent effect on the gingiva to enable a clear and detailed record of the relationship between the prepared tooth and the surrounding tissues to be obtained.

### Indications

For temporary lateral retraction of the free gingiva and, in the case of a healthy periodontium, the drying of the sulcus to improve the detail of the subsequent impression.

### Contra-Indications

Contains kaolin and aluminium chloride. Do not use the paste in patients with hypersensitivity to these substances. Do not use in the case of exposed bone, advanced periodontal disease or exposed furcations.



### Side effects

There are no known side effects to date. Isolated cases of hypersensitivity to components of the material cannot be excluded.

### Preparation, Dispensing & Application

1. Prepare the tooth as appropriate for the case concerned.
2. Immediately before use remove a compute from an unopened sachet. Place into a suitable compute applicator. Computes fit most such applicators.
3. Bend the tip of the compute to the optimal position for access.
4. Extrude an initial amount of the paste onto the mixing pad & discard.
5. Rinse the sulcus with water and dry with compressed air. Before application of the Retraction Paste dry the preparation area thoroughly, such as with suction and cotton rolls.
6. Apply the paste slowly into the sulcus round the preparation until it is completely filled. Use compression caps or other compression products as required.



7. Allow paste to take effect for approximately 2 minutes in order to enable gingival retraction. During this period it is essential that the area treated is kept completely dry.

**Caution:** Do not leave in the mouth longer than recommended since this may lead to irritation of the gingiva.

8. Using a water/air mixture and suction device completely remove all paste from the mouth and especially in and around the gingival sulcus. The highly visible colour of the paste facilitates the checking of this by eye.

### Cord Technique


If required by the clinical situation Doric Gingival Retraction Paste may be used in combination with retraction cords.

### Technical information

Working time: approximately 2 minutes.  
Composition: kaolin, aluminium chloride hexahydrate, water, pigments, flavouring.

### Storage

Store out of direct sunlight and at temperatures between 15°C & 25°C.

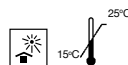

### Health & Safety

Avoid contact with eyes! In case of accidental contact, rinse immediately with plenty of water and consult a doctor if necessary. Do not swallow the paste! In case of accidental ingestion consult a doctor if necessary. Keep away from children.



Refer to Safety Data Sheet.

### Lot Number & Expiry Date

The lot number and expiry date are shown on all boxes. Do not use after the expiration and after use dispose of any material remaining in the compute.

This product is specifically formulated for use in dentistry.

*Doric is an internationally registered trademark of Davis Schottlander & Davis Ltd.*

