

Doric Universal Adhesive Instructions for Use

INDICATIONS

Doric Universal Adhesive is designed to provide adhesion of addition cured silicones (VPS) to impression trays that have no or only minimal retention. It provides reliable retention to metal and plastic trays. When using plastic trays always ensure that the impression tray is compatible with the adhesive.

CONTRA-INDICATIONS

Do not use when allergies against any of the components are known.

APPLICATION

1. Ensure that the impression tray is clean, grease free and dry.
2. Brush the Doric Universal Adhesive onto the inside of the tray so that there is a thin and uniform film.
3. Leave the adhesive on the tray for 3 minutes to ensure a good bond to the tray.
4. Load the addition cured silicone into the tray within 30 minutes of application to ensure that the adhesive provides its full bond strength.

Note: Always apply and allow solvents to evaporate in a well ventilated room.

STORAGE

Store at normal room temperature ($\leq 25^{\circ}\text{C}$). Close bottle immediately after use to prevent contamination or evaporation of the solvent. Do not use after expiry date shown on the container.

COMPOSITION

Silicone resin, solvent (volatile paraffin).

HEALTH & SAFETY



This item is flammable. Do not inhale, do not swallow. Avoid contact with skin or eyes. In case of accidental contact wash skin with soap and water. In case of contact with eyes rinse immediately with plenty of water and consult a doctor.

Refer to safety data sheet.

In case of an allergic reaction wash with soap and water. Discontinue further use or wear gloves as necessary. Use in a well ventilated room and avoid inhaling vapour or eye contact.

This product is specifically formulated for use in dentistry.

Doric is an internationally registered trade mark of Davis Schottlander & Davis Limited.