


H C STENTS

DIRECTIONS FOR USE

Immerse the required quantity of Stents Composition in hot water at a temperature of 55° Centigrade.

When softened, knead it thoroughly with fingers and place in a dry Impression Tray. Before inserting in the mouth, pour a little cold water over the back of the Impression Tray. Glaze the surface with dry heat (over a spirit lamp or Bunsen), being careful not to use too much heat and quickly insert in the mouth. After removal, put into cold water.

Caution: undue heat applied to this compound will cause it to deteriorate.

Single use. 

This product is specifically formulated for use in dentistry.

Davis Schottlander & Davis Ltd,
Letchworth, Herts, England
+44 (0)1462 480848 www.schottlander.com



W5490/0605