

# R-Motion RECIPROCATION



**schottlander**

**ONE SEQUENCE TO  
FIT MOST CASES**

## GLIDE PATH

**R-Motion  
Glider C**



## SHAPING

**R-Motion  
25**



## MINIMALLY INVASIVE - LESS REMOVAL OF SOUND DENTINE

Unlike other reciprocating file systems that are variable in taper and so remove excessive coronal dentine, the improved design of **R-Motion** files allows all shaping files to be made with a constant taper. This minimises the removal of sound dentine while allowing a generous apical preparation for improved irrigation.

## SMOOTH TO USE

Enjoy reciprocation technology that feels smooth and brings a total peace of mind.

## SIMPLE + COST EFFECTIVE

Thanks to the agile design a glide path and a single shaping Instrument is sufficient in most cases.

**Simplicity itself.**

Instruments to be used in reciprocating, counter clockwise, mode.

**30%  
DISCOUNT**

**Only £7.92  
per file**

### R-Motion

Pack contains three instruments of a type.  
List Price £33.95 per pack.

**Offer Price £23.76  
per pack  
(£7.92 per file)**

Instrument	21mm Code	25mm Code	31mm Code
<b>R-MOTION GLIDER C</b> (15/.03v)	-	937-71	-
<b>R-MOTION 25</b> (25/.06)	936-13	936-53	936-93
<b>R-MOTION GLIDER</b> (15/.03v)	-	937-51	-
<b>R-MOTION 20</b> (20/.04)	-	936-03	-
<b>R-MOTION 30</b> (30/.04)	935-14	935-54	935-94
<b>R-MOTION 40</b> (40/.04)	935-16	935-56	935-96
<b>R-MOTION 50</b> (50/.04)	935-18	935-58	935-98

### GLIDE PATH FILE

Recommended for narrow and/or more curved canals where more rigidity is required to reach the apex

**R-Motion  
Glider C**



15/.03v  
up to WL

### SHAPING FILE

Final file for standard canals  
(most cases)

**R-Motion  
25**



25/.06  
up to WL

### OPTIONAL FILES FOR NARROW OR LARGE CANALS

**R-Motion Glider** 15/.03v up to WL

Can be used **instead** of the R-Motion Glider C for wider and straighter canals if preferred by the dentist.

**R-Motion 20** 20/.04 up to WL

Intermediate file only required for narrow canals.  
Use **before** main shaping file if required.

**R-Motion 30** 30/.04 up to WL

Can be used **instead** of R-Motion 25 for a larger apical preparation and less coronal, dentine removal, if preferred by the dentist.

**R-Motion 40** 40/.04 up to WL

For use **only** when needed in larger canals.  
Use **after** main shaping file.

**R-Motion 50** 50/.04 up to WL

For use **only** when needed in larger canals.  
Use **after** main shaping file.

Schottlander Freephone  
**0800 97 000 79**  
sales@schottlander.co.uk  
**www.schottlander.com**

Offer ends 30th June 2023.  
Price includes delivery but excludes VAT.

**SAMPLE PACK  
WORTH £30**

Contains two instruments  
and one 3D-Training Tooth

